

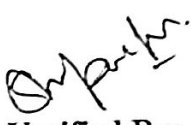
Calibration: To calibrate the Instrument using calibrator (ABX Minocal) and verify the same.

Procedure: Go to Quality Assurance icon on main screen and then Calibration icon. Run Calibrator (ABX Minocal) 6 times, without taking the values of first run, calibrate the instrument using average of the last 6 runs. Run Calibrator as a sample to verify the calibration.
Lot: CX436; Expiry: 05/08/2019.

Serial No: 811PES15121

| Parameter | Target Value (As per Kit Insert) | Mean Value | Observed CV% | Acceptance CV% | Comments |
|-----------|--|---------------|-----------------|-------------------|----------|
| WBC | 9.0 | 9.38 | 0.37 | <2% | PASSED |
| RBC | 4.59 | 4.66 | 0.4 | <2% | PASSED |
| HGB | 13.7 | 13.86 | 0.45 | <1% | PASSED |
| HCT | 37.3 | 38.02 | 0.62 | <2% | PASSED |
| PLT | 250 | 255.62 | 1.57 | <5% | PASSED |


Conducted By:


Verified By: