# Byovet Medical & Technologies Pvt. Ltd.,

Redefining Veterinary Diagnostics

### Record of Analyzer Insatallation, Check, Accept and training

**User Name** 

**IQURE BIOMEDICAL SYSTEMS PVT LTD** 

**Address** 

13/1, 2ND FLOOR, 1ST MAIN ROAD, KHM BLOCK, GANGANAGAR, RT

NAGAR, BANGALORE-560032

**Product Name** 

HAEMATOLOGY ANALYSER

Model

SMART DX

**Serial Number** 

48600003

Qty

1

Date of Installation 24.01.2021

**Warranty Period** 

24.01.2021 TO 23.01.2022

**Check and Accept** 

PRODUCT ACCESSORIES, QUALIFICATION MATERIALS, MANUALS,

CERTIFICATES AND OTHERS ARE COMPLETE.

**Training** 

TRAINER:

JAGADISH

**DATE OF TRAINING:** 

25.01.2021

**Customer Opinion** 

INSTALLATION DONE SUCCESSFULLY

**Date** 

25.01.2021

Signature of Check and accept

**Warranty Notes:** 

1. This record also used as warranty certificate. Please ke

2. This record is in duplicate. The first one maintained by the installation engineer, and the second one is maintained by the customer.

3. Please reffer to manuals about warranty service.



## Byovet Medical & Technologies Pvt. Ltd.,

Redefining Veterinary Diagnostics

#### SMART DX HAEMATOLOGY SYSTEM OPERATIONAL QUALIFICATION

**Customer Name** 

**IQURE BIOMEDICAL SYSTEMS PVT LTD** 

13/1, 2<sup>ND</sup> FLOOR, 1<sup>ST</sup> MAIN ROAD, KHM BLOCK, GANGANAGAR,

RT NAGAR, BANGALORE, KARNATAKA-560032

THE FOLLOWING CHECKS AND SETUPS ARE CARRIED OUT FOR BYOVET SMART DX HEMATOLOGY ANALYZER FOR OPERATIONAL QUALIFICATION.

1. DEFINE SETUP AS PER LABORATORY REQUIREMENTS

	• SAMPLE	
	SYSTEM OPTIONS     PRINTER AND MEMORY	
2.	PERFORM DIAGNOSTIC (SLOW SPEED AND HIGH SPEED)	VEC
	CHECK FOR SHEAR VALVE	YES
3.	PERFORM DIAGNOSTIC CHECK FOR TUBINGS AND VALVE	VEC
	TITLE OF THE PARTIES THE CHECK FOR TODDINGS AND VALVE	YES
4.	PERFORM BACKGROUND TEST FOR DILUENT AND LYSE	YES
5.	PERFORM 2 POINT PRIMING SOLUTION	YES
6.	PERFORM CONDITIONING BY USING FRESH WHOLE BLOOD	YES

7. PERFORM SAMPLE ASPIRATION IN BACKGROUND CHECK YES

8. PERFORM FULL 2 POINT CALIBRATION YES (CALIBRATION PERFORMED BY THE INSTRUMENT ITSELF FOR REGULAR USAGE) 9. HANDING INSTRUMENT AND USER TRAINING

10. CHECKED FOR TRUE VALUE CHECK (QC LOW) YES (REFER PERFORMANCE QUALIFICATION FOR DETAILS)

11. THE SYSTEM IS READY FOR SPECIFIC USAGE. YES

Company Name	BYOVET MEDICAL TECHNOLOGIES PVT LTD	Page 2 of 3
<b>Equipment Name</b>	SMART DX	Serial No: 48600003
	Operational Qualification	



YES

YES

### Byovet Medical & Technologies Pvt. Ltd.,

Redefining Veterinary Diagnostics

#### SMART DX Haematology System Performance Qualification

Following procedure was carried out as part of the performance Qualification:-

#### 1. Calibration of Assay

Analytes Calibrated and checked for WBC, RBC, HGB, MCV, PLT, MPV, RDW-CV, RDW-SD

Checked and found all calibration are valid Curve.

#### 2. Internal Quality Control Performance.

RANDOX Haematology Control Checked and found Controls are within the acceptable limits.

#### 3. Precision Study

Run Precision for **WBC**, **RBC**, **HGB**, **MCV**, **PLT**, **MPV**, **RDW-CV**, **RDW-SD**Assays of 10 replicates was carried out, CV% obtained
Are within the acceptable limit for the assay
See print outs attached.

#### 4. Carryover Check

Performed the Carryover study in **SMART DX** as per BCI protocols See printout attached

#### 4. BYOVET Methodology & Range

See print outs attached

Company Name	BYOVET MEDICAL TECHNOLOGIES PVT LTD	Page 1 of 3
<b>Equipment Name</b>	SMART DX	Serial No: 48600003
	Performance Qualification (Pe	Q) ct Pvt

90:	2021-01-25 13:06	blood	Mode: whole bl		Next ID: 431797	perator: JAGADISH
	nar Nar		Export		Save	
	V		10 (1) (1) (1) (1) (1) (1) (1) (1) (1) (1)			
	85.0	100.0	80.0	85.0	100.0	Cal%
	****	****	****	****	**************************************	New Cal%(%)
1503	19.6004	0.2160	0.1766	0.0437	01567	SD
	232	90.9	14.1	4.23	8.3	Mean
	208	90.8		4.26	8.4	5
20	221	90.8	140	4.19	83	9
1991	215	907		A N N	8.7	
	225	90.6	129	4.16	8.0	
	234	91.1		4.24	88	
	700	91.0		4.22	8.4	
l g	220			4.29	8.3	4
	720		14.4	4.25	8.3	<b>.</b>
	230			<b>4.20</b>	<b>68</b>	2
	0/7		The first of the second of the	4.30	8.4	<u>.</u>
		the state of the s		And the state of t		Ref. Value
1	PLT	MCV	HGB	RBC	WBC	
J				112	Lot 1 154312	alibrator L
		A Section of the sect				
-	The second secon					The state of the s

Ì,

