### Byovet Medical & Technologies Pvt. Ltd.,

Redefining Veterinary Diagnostics

### Record of Analyzer Insatallation, Check, Accept and training

**User Name** 

**IQURE BIOMEDICAL SYSTEMS PVT LTD** 

**Address** 

13/1, 2ND FLOOR, 1ST MAIN ROAD, KHM BLOCK, GANGANAGAR, RT

NAGAR, BANGALORE-560032

**Product Name** 

HAEMATOLOGY ANALYSER

Model

SMART DX

**Serial Number** 

48600003

Qty

1

Date of Installation 24.01.2021

**Warranty Period** 

24.01.2021 TO 23.01.2022

**Check and Accept** 

PRODUCT ACCESSORIES, QUALIFICATION MATERIALS, MANUALS,

CERTIFICATES AND OTHERS ARE COMPLETE.

Training

TRAINER:

JAGADISH

DATE OF TRAINING:

25.01.2021

**Customer Opinion** 

INSTALLATION DONE SUCCESSFULLY

**Date** 

25.01.2021

Signature of Check and accept

**Warranty Notes:** 

1. This record also used as warranty certificate. Please ke

2. This record is in duplicate. The first one maintained by the installation engineer, and the second one is maintained by the customer.

3. Please reffer to manuals about warranty service.



#### Byovet Medical & Technologies Pvt. Ltd.,

Redefining Veterinary Diagnostics

#### SMART DX HAEMATOLOGY SYSTEM OPERATIONAL QUALIFICATION

**Customer Name** 

**IQURE BIOMEDICAL SYSTEMS PVT LTD** 

13/1, 2<sup>ND</sup> FLOOR, 1<sup>ST</sup> MAIN ROAD, KHM BLOCK, GANGANAGAR,

RT NAGAR, BANGALORE, KARNATAKA-560032

THE FOLLOWING CHECKS AND SETUPS ARE CARRIED OUT FOR **BYOVET SMART DX HEMATOLOGY ANALYZER** FOR OPERATIONAL QUALIFICATION.

1.	DEFINE SETUP AS PER LABORATORY REQUIREMENTS	YES
	• Q.C • SAMPLE	
	SYSTEM OPTIONS	
	PRINTER AND MEMORY	
2.	PERFORM DIAGNOSTIC (SLOW SPEED AND HIGH SPEED)	YES
	CHECK FOR SHEAR VALVE	
3.	PERFORM DIAGNOSTIC CHECK FOR TUBINGS AND VALVE	YES
4	PERFORM BACKGROUND TEST FOR DILUENT AND LYSE	YES
4.	PERFORM BACKGROUND TEST FOR DIEGENT AND 2132	125
5.	PERFORM 2 POINT PRIMING SOLUTION	YES
6.	PERFORM CONDITIONING BY USING FRESH WHOLE BLOOD	YES
7.	PERFORM SAMPLE ASPIRATION IN BACKGROUND CHECK	YES
8.	PERFORM FULL 2 POINT CALIBRATION	100 V X V 100
0.	(CALIBRATION PERFORMED BY THE INSTRUMENT ITSELF FOR REGULAR USAGE)	YES
	TO A THIRD COLOR TO A THIRD	VEC
9.	HANDING INSTRUMENT AND USER TRAINING	YES
10.	CHECKED FOR TRUE VALUE CHECK (QC LOW)	YES
	(REFER PERFORMANCE QUALIFICATION FOR DETAILS)	123
	THE SYSTEM IS READY FOR SPECIFIC USAGE.	YES
11.	THE STOTEM TO KLADT TOK OF TOTAL TO GOTTON.	

Company Name	BYOVET MEDICAL TECHNOLOGIES PVT LTD	Page 1 of 1	
Equipment Name	SMART DX	Serial No: 48600003	
Equipment Name	OPERATIONAL QUALIFICATION		



# Byovet Medical & Technologies Pvt. Ltd.,

\_\_Redefining Veterinary Diagnostics\_

# SMART DX HAEMATOLOGY SYSTEM PERFORMANCE QUALIFICATION

Following procedure was carried out as part of the performance qualification:

### 1. Calibration of Assay

- Analytes calibrated and checked for WBC, RBC, HGB, MCV, PLT, MPV, RDW-CV, RDW-SD
- b. Checked and found all calibration are valid curve.

## 2. Internal Quality Control Performance

- a. RANDOX Hematology Control
- b. Checked and found controls are within the acceptable limits.

### 3. Precision Study

- Run precision for WBC, RBC, HGB, MCV, PLT, MPV, RDW-CV, RDW-SD
- Assays of 10 replicates was carried out, CV% obtained are within the acceptable limit for the assay
- See print outs attached

	DEDECOMANCE CHAITETCATTON (PO)	
Serial No: 48600003	SMART DX	Equipment Name
Page 1 of 3	BYOVET MEDICAL TECHNOLOGIES PVT LTD	Company Name



051-01-52 13:0	S boo	Mode: whole bl		Vext ID: 431797	N HSIOA6	Al:
		nod	e P	VIEZ		**************************************
				Nice of the second seco	A CONTRACTOR OF THE PROPERTY O	
0.28	100°0	0.08	0.28	0'00T	We will not be a second of the	%
****	****	***	***	****	(%)	1% E
7009'6T	0917.0	99/T'0	7.540.0	L9STO	78 No. 1	C
737	6.06	I.AI	4.23	£.8		ue
208	8'06	THE	927	<b>78</b>	Contract of the special state	(
721	8.06	14.0	6T#	£.8		
<b>213</b>	Z 05	過世世で10世紀	EZ V	<u> </u>		
220	9.06	6'EL	91.4	0.8		
23d	T:16	1-0.41	72.7	<b>E8</b>	A PARTIE	
. 590	0.1e	ן דאדן	4.22	78		
9EZ	- Z.12	E'FT	627	E'8		
230	Z'T6	777	4'52'm	E.8	A	
08Z	6 06	0.41	d. 20	F8 3 554	规定《	
770	Z'06	THE	. <b>08</b> t	8.4	and and a second	
E E E NA TRANS				引。其時心脈動性質		ənį
	MCV	HeB		MBC		

---

0.51 -01-25 13:0	poo	lode: whole blo	N The Later of the	767184:0	JI 1x9M	HSIGAĐAL
M Miles and		port	EX	evel		""。
		(1) (1) (1) (1) (1) (1) (1) (1) (1) (1)				
T00.0	0.00£	0'00T	0.28	0.00L		% E
*****	****	****	****	*****		(%)% e_
3.5547	₽78E.0	0.1032	₱009.61	0.2160		2D
4.85	L'6	1.8	732	6.06		esu
35.2	0.6	0.8	807	8.06	7	01
0.94	I.O.I	Z.8	ZZZ	8.06	2	6
9.04	9'6	2.8		L'06	* 2	8
Z.Z.	9.6	0.8	550	9.06		L. S. L.
9'07~ -	5'6	T.8	- S34 · .	TT6	7	
6.75	5.6	£.8	590	0.19	2	S
9.04	0:0T	T.8		Z'T6		<b>b</b>
Z.2£	S'6	2.8	<b>Z30</b>	2.19		3
ZSE	9.6	* T.8	730	~ 6'0 <b>6</b> ~	2	7
6'ZE	LOI	0.8	270	<u>Z</u> .06	2	Value T
		1.00	7 7 4 4 7 4 4 5	MCV	The Sales	