

GSTIN:- 08CIUPP8490B1ZL

SP DIAGNOSTIC

Sales & Service Medical & Laboratory Instruments.

Contec BC 300 (Semi auto chemistry Analyzer)
PPM Checklist

Purpose

This document provides a list of scheduled preventive maintenance procedures to be performed at the recommended frequencies.

Institution Name:	Vishwakarma Pathology Lab		
System Serial No.:	1906050000 9	Software version and install date:	17.03.01.1.2.3, 11/2019
Frequency:	Annually		
PPM Date	17/02/2021	PPM Due Date	16/02/2022
Warranty:	<input type="checkbox"/>	Contract:	<input type="checkbox"/>
		On Demand:	<input type="checkbox"/>

Disclaimer

The installation and service of equipment described herein is to be performed by qualified personnel.

Error! Reference source not found.	
Check the State of the Analyzer	
Run a Self-Test and check DC levels.	<input checked="" type="checkbox"/>
Perform the Peristaltic pump calibration	<input checked="" type="checkbox"/>
Check the Temperature Control Unit.	<input checked="" type="checkbox"/>
Drain the Flow Cell.	<input checked="" type="checkbox"/>
Check the gain of all filter	<input checked="" type="checkbox"/>
Inspect the Flow cell and Tubing	
Check for dirt or deposits and any sign of spills in Flow cell	<input checked="" type="checkbox"/>
Clean the flow cell.	<input checked="" type="checkbox"/>
Check aspiration tubing, Replace as necessary.	<input checked="" type="checkbox"/>
Check drainage tubing. Replace as necessary.	<input checked="" type="checkbox"/>
Run blood samples in all different mode, check results and run a Repeatability test.	<input checked="" type="checkbox"/>
Preventive Maintenance final status	Pass

Date	Signature (FSE)	Print name



G-159 SECTOR-G SHASTRI NAGAR, NEAR ST. PAULS SCHOOL, JODHPUR (RAJ.) 342001
EMAIL: SPDIAGNOSTIC2704@YAHOO.COM
Mob: +91-8233194862, +91-7791958626