



CL MICROMED PVT. LTD.

T-17, OKHLA INDUSTRIAL AREA PHASE-2, NEW DELHI-110 020

CIN : U74899DL1987PTC027941

TEL : 91-11-26387262, 26387263

FAX : 91-11-26387264

e-mail : info@clmicromed.com

www.clmicromed.com

TECHNICAL / SERVICE REPORT

Date : 24-09-2021.....

INSTALLATION WARRANTY PERIOD CALL SERVICE CONTRACT CALL CHARGED CALL COURTESY

Customer's Name : Fatima Hospital (Laboratory).....

Address : Mother Teresa Road, Padri Bazar
Gosakhpur (U.P.).....

Contact Person : Laboratory I/c...... Telephone : 0551-2281862.....

Instrument : Electrolyte Analyser.....

Model : COMBILINE..... Make : Eschweiler GmbH..... Serial No.: CL 0030
Germany

Nature of Fault : Routine maintenance & calibration service......

Description of Job : Preventive maintenance carried out. Performed cleaning & activation
of all the sensors. Calibration done and sensor parameters found OK.
checked Q.C. results, found correct & precise. The equipment
working satisfactorily.

1. Fault Rectified... yes..... 2. Job to be continued... N/A..... 3. Preventive Maintenance... yes.....

PART REPLACED Form Our Stock From Customer's Inventory

Sr. No.	Description	Qty.
1.	Ref. Membrane (FOC)	1
2.	Set tubing	1

Engineer's Remarks..... /

Customer's Comments (if any).....

Instrument is working satisfactorily.

Engineers Name : Alok Verma

Contact No. : 9415021528

Engineer's Signature A.V.

Customer's Signature

Date :

ESCHWEILER, COMBI LINE

SENSORPARAMETER

DATE 09:51 24.09.21

K - SENSOR

TIME	SLOPE	EC1	EC2
0906	97	280	-298
0733	99	291	-297
0725	99	284	-301
0523	99	275	-310
0357	99	247	-341
2340	00	255	-333
MEAS. [mV]		19	

NR - SENSOR

TIME	SLOPE	EC1	EC2
0906	105	182	3
0733	104	178	2
0726	106	183	4
0523	107	197	10
0357	06	241	77
2340	103	183	6
MEAS. [mV]		86	

GR - SENSOR

TIME	SLOPE	EC1	EC2
0906	89	1020	701
0733	89	1028	757
0726	80	1021	758
0523	80	1021	752
0357	80	003	732
2340	91	1012	746
MEAS. [mV]		012	

DEPT OF LAB
SERIAL NO. C10030
CAL. DATE 24/09/2021
VALID UPTO 23/03/2022



START BY	DEFLECTION
K == ACTIVE	
NO == ACTIVE	
CL == ACTIVE	MED
	QC
	RESP. GAS

16:22 CAL. 29 MIN

Control panel with four horizontal sliders and a numeric keypad:

- Slider 1 (top): 1 2 3
- Slider 2: 4 5 6
- Slider 3: 7 8 9
- Slider 4 (bottom): C 0 .

Below the keypad is a yellow circular button with a left-pointing arrow.

ESCHWEILER

combi  line

