Registered Office:

BioSystems Diagnostics Pvt. Ltd.,

A4, SIPCOT Industrial Park, Irungattukottai, Thandalam Village, Sriperumbudur - 602105. Ph: +91 - 44 - 3034 1911 / 12 /13 E-mail: customersupport@biosystems.in

Website: www.biosystems.in CIN – U29297TN2003PTC051492



A15C - Fully Auto Biochemistry Analyzer

Installation Qualification Certificate

For

Hannah Joseph Hospital

BioSystems Diagnostics Pvt. Ltd



A4, SIPCOT Industrial Park, Irungattukottai, Thandalam Village, Sriperumbudur - 602105. Ph: +91 - 44 - 3034 1911 / 12 /13 E-mail: customersupport@biosystems.in

E-mail: customersupport@biosystem Website: www.biosystems.in CIN – U29297TN2003PTC051492



CERTIFICATE OF INSTALLATION

Instrument Name: A15C Auto Analyzer

Serial Number : 83105C0880

Customer : Madurai – Tuticorin Ring, Chithamani – Madurai -625009

The undersigned performer certifies that the installation Qualification Protocol has been successfully completed for the instrument stated above.

Engineer

Signature

Name

Designation : Company

: Mr. Alaguraj: Service Engineer

: BioSystems Diagnostics Pvt. Ltd

Customer

Signature :

Name : Dr. Abarna Pandey Designation : Quality Manager

Company : Hannah Joseph Hospital

Corporate Office:

BioSystems S.A. Costa Brava 30, 08030 Barcelona (Spain) Tel +34-93 311 00 00, Fax+34-93 346 77 99



Registered Office: BioSystems Diagnostics Pvt. Ltd., A4, SIPCOT Industrial Park, Irungattukottai,

Thandalam Village, Sriperumbudur - 602105. Ph: +91 - 44 - 3034 1911 / 12 /13 E-mail: customersupport@biosystems.in Website: www.biosystems.in CIN – U29297TN2003PTC051492



INSTALLATION REPORT

Customer Name : Hannah Joseph Hospital

Department : Bio-chemistry

Contact Person : Dr. Abarna Pandey

Instrument Model : A15C

Serial Number : 83105C0880

Date of Installation : 19-07-2021

The System was installed along with the necessary accessories. All basic requirements and adjustments were checked and are found to be satisfactory Preliminary customer training was provided and standardization of all parameters was done. The Instrument is found to be working satisfactorily.

Corporate Office: BioSystems S.A. Costa Brava 30, 08030 Barcelona (Spain) Tel +34-93 311 00 00, Fax+34-93 346 77 99



BioSystems Diagnostics Pvt. Ltd.,

A4, SIPCOT Industrial Park, Irungattukottai, Thandalam Village, Sriperumbudur - 602105. Ph: +91 - 44 - 3034 1911 / 12 /13

E-mail: customersupport@biosystems.in Website: www.biosystems.in CIN - U29297TN2003PTC051492



INSTRUMENT SETUP:

- Instrument and computer installed
- Printer Installed
- User software 5.4 installed
- All necessary cable connections given

OPERATION INSPECTION:

- No air bubbles found in the Tubing
- Temperature of the Needles/Reaction & Rotor Chamber are within range
- No unusual noises produced in the system
- Warm up procedure within time limit
- Internal Dosing of all Needle's were fine
- External Dosing of all Wash Station is perfect
- Conditioning of Machine with Air/Wash solution/Distilled water done

For Biosystems Diagnostics Pvt. Ltd.,

Mr. Alaguraj

Service Engineer

Corporate Office:

BioSystems S.A. Costa Brava 30, 08030 Barcelona (Spain) Tel +34-93 311 00 00, Fax+34-93 346 77 99

BioSystems Diagnostics Pvt. Ltd., A4, SIPCOT Industrial Park, Irungattukottai, Thandalam Village, Sriperumbudur - 602105.

Ph: +91 - 44 - 3034 1911 / 12 /13 E-mail: customersupport@biosystems.in Website: www.biosystems.in CIN – U29297TN2003PTC051492



INSTALLATION QUALIFICATION

Customer Name : CM Best Hospitals

Address : Hannah Joseph Hospital

Madurai – Tuticorin Ring, Chithamani

Madurai -625009

Instrument Model : A15C

Serial No. : 83105C0880

Operational requirement for the Instrument:

Requirement	Acceptable Range	Observed Range
Input Voltage	220 <u>+</u> 10 V	221 V
Line Frequency	50 or 60 Hz	50 Hz
Ambient Temperature	18 - 30 deg. C	22 deg. C
Distill Water	Type II NCCLS	Available

For Biosystems Diagnostics Pvt. Ltd



Corporate Office: BioSystems S.A. Costa Brava 30, 08030 Barcelona (Spain) Tel +34-93 311 00 00, Fax+34-93 346 77 99 http://www.biosystems.es



BioSystems Diagnostics Pvt. Ltd.,

A4, SIPCOT Industrial Park, Irungattukottai, Thandalam Village, Sriperumbudur - 602105. Ph: +91 - 44 - 3034 1911 / 12 /13 E-mail: customersupport@biosystems.in

E-mail: customersupport@biosyster Website: www.biosystems.in CIN – U29297TN2003PTC051492



A15C - Fully Auto Biochemistry Analyzer

Operational Qualification

For

Hannah Joseph Hospital

BioSystems Diagnostics Pvt. Ltd

Corporate Office:

BioSystems S.A. Costa Brava 30, 08030 Barcelona (Spain) Tel +34-93 311 00 00, Fax+34-93 346 77 99



Registered Office:

Registered Office:
BioSystems Diagnostics Pvt. Ltd.,
A4, SIPCOT Industrial Park, Irungattukottai,
Thandalam Village, Sriperumbudur - 602105.
Ph: +91 - 44 - 3034 1911 / 12 /13
E-mail: customersupport@biosystems.in

Website: www.biosystems.in CIN – U29297TN2003PTC051492



OPERATIONAL QUALIFICATION

- System Decontamination 1.
- Maintenance Procedures 2.
- System Performance check procedures 3.
- Customer Training Operation and Maintenance 4.

ISO 13485

BioSystems Diagnostics Pvt. Ltd., A4, SIPCOT Industrial Park, Irungattukottai, Thandalam Village, Sriperumbudur - 602105.

Ph: +91 - 44 - 3034 1911 / 12 /13 E-mail: customersupport@biosystems.in Website: www.biosystems.in CIN – U29297TN2003PTC051492



CERTIFICATE OF OPERATION

Instrument Name : A15C Auto Analyzer

Serial Number : 83105C0880

Customer : Hannah Joseph Hospital

Madurai – Tuticorin Ring, Chithamani

Madurai -625009

The undersigned performer certifies that the Operational Qualification Protocol has been successfully completed for the instrument stated above.

Engineer

Signature :

Name : Mr. Alaguraj
Designation : Service Engineer

Company : BioSystems Diagnostics Pvt. Ltd

Customer

Signature :

Name : Dr. Abarna Pandey Designation : Quality Manager

Company : Hannah Joseph Hospital

Corporate Office:

BioSystems S.A. Costa Brava 30, 08030 Barcelona (Spain) Tel +34-93 311 00 00, Fax+34-93 346 77 99



Registered Office:
BioSystems Diagnostics Pvt. Ltd.,
A4, SIPCOT Industrial Park, Irungattukottai,
Thandalam Village, Sriperumbudur - 602105.
Ph: +91 - 44 - 3034 1911 / 12 /13
E-mail: customersupport@biosystems.in
Website: www.biosystems.in
CIN - U29297TN2003PTC051492



SYSTEM PERFORMANCE CHECK PROCEDURE

SYTEM CHECK	PERFORMANCE	
System Decontamination	Done	
LED Photometric Check	Done	
Warm Up	Done	
Mechanical Adjustments	Done	
Temperature Check	Done	
Internal/External Dosing Check	Done	

Customer Signature:



A4, SIPCOT Industrial Park, Irungattukottai, Thandalam Village, Sriperumbudur - 602105. Ph: +91 - 44 - 3034 1911 / 12 /13

E-mail: customersupport@biosystems.in Website: www.biosystems.in CIN - U29297TN2003PTC051492



OPERATIONAL QUALIFICATION

System Certification:

Study data has determined that the System described in this document either meets all criteria outlined in this Operational Qualification Protocol, or exceptional conditions have been identified and documentation included. Exceptional conditions, if any have been addressed. The system is ready for specified usage.

Protocol Performed By: BioSystems Diagnostics Pvt. Ltd.,

> Name : Mr. Alaguraj

Title : Service Engineer

Company: Biosystems Diagnostics Pvt Ltd

Customer Authorization: Hannah Joseph Hospital

Name: Dr. Abarna Pandey

Title: Quality Manager

CUSTOMER SIGNATURE

DATE: DATE:

Corporate Office: **BioSystems S.A.** Costa Brava 30, 08030 Barcelona (Spain) Tel +34-93 311 00 00, Fax+34-93 346 77 99



BioSystems Diagnostics Pvt. Ltd.,

A4, SIPCOT Industrial Park, Irungattukottai, Thandalam Village, Sriperumbudur - 602105. Ph: +91 - 44 - 3034 1911 / 12 /13

E-mail: customersupport@biosystems.in Website: www.biosystems.in CIN – U29297TN2003PTC051492



A15C - Fully Auto Biochemistry Analyzer

Performance Qualification

For

Hannah Joseph Hospital

BioSystems Diagnostics Pvt. Ltd



BioSystems S.A. Costa Brava 30, 08030 Barcelona (Spain) Tel +34-93 311 00 00, Fax+34-93 346 77 99



Registret Opt. Residence Pvt. Ltd., A4, SIPCOT Industrial Park, Irungattukottai, Thandalam Village, Sriperumbudur - 602105. Ph: +91 - 44 - 3034 1911 / 12 /13 E-mail: customersupport@biosystems.in

Website: www.biosystems.in CIN – U29297TN2003PTC051492



A15C Auto Analyzer

PERFORMANCE QUALIFICATION:

The following parameters were checked and found to be within acceptable limits.

S.No	Parameters	Result	Limit
1.	Dispensing Ratio	15.10	14.8 - 17.2
2.	Photometric noise	0.43	< 3.0
3.	Darkness count	4021	3000 - 5000
4.	Slope	0.0031	+/- 0.02





Corporate Office: BioSystems S.A. Costa Brava 30, 08030 Barcelona (Spain) Tel +34-93 311 00 00, Fax+34-93 346 77 99 http://www.biosystems.es

BioSystems Diagnostics Pvt. Ltd., A4, SIPCOT Industrial Park, Irungattukottai, Thandalam Village, Sriperumbudur - 602105.

Ph: +91 - 44 - 3034 1911 / 12 /13 E-mail: customersupport@biosystems.in Website: www.biosystems.in CIN – U29297TN2003PTC051492



CERTIFICATE OF PERFORMANCE

Instrument Name : A15C Auto Analyzer

Serial Number : 83105C0880

Customer : Hannah Joseph Hospital

Madurai – Tuticorin Ring, Chithamani

Madurai -625009

The undersigned performer certifies that the Performance Qualification Protocol has been successfully completed for the instrument stated above.

Engineer

Signature :

Name : Mr. Alaguraj
Designation : Service Engineer

Company : BioSystems Diagnostics Pvt. Ltd

Customer

Signature

Name : Dr. Abarna Pandey Designation : Quality Manager

Company : Hannah Joseph Hospital

Corporate Office:

BioSystems S.A. Costa Brava 30, 08030 Barcelona (Spain) Tel +34-93 311 00 00, Fax+34-93 346 77 99

