



VECTOR BIOTEK PVT. LTD.

FACTORY & REGISTERED OFFICE :
425, NEW GIDC, KABILPORE, NAVSARI - 396 424 (GUJ) INDIA
Toll Free : 1800-270-1027 | E-mail : vector@beaconindia.com
Website : www.vectorbiotekindia.com | CIN No. : U33112GJ2002PTC040978

Performance Qualification

MODEL: B AUTO 200

Sr. No: AES2K0WH01023

Version: V4.8.3.0B3

Date: 08-06-2022

PQ is performed at

Reliable Diagnostic centre, Chomu, Jaipur, Rajasthan.

According to the assay results that are comprised in this document the B AUTO 200 system can be approved for the routine operation.

System released for routine operation.

PERFORMANCE CALIBRATION

LOT No: 26481

TEST	TARGET	RANGE	RESULT
GLU	79.45	71.50-87.40	82.07
CHO	280.50	236-325	268.1
ALT	25.35	18.60-32.10	21.62
AST	40.90	35.90-45.90	36.71
ALP	166.30	135.30-197.30	197.2
BIL-D	0.32	0.20-0.44	0.38



VECTOR BIOTEK PVT. LTD.

FACTORY & REGISTERED OFFICE :
425, NEW GIDC, KABILPORE, NAVSARI - 396 424 (GUJ) INDIA
Toll Free : 1800-270-1027 | E-mail : vector@beaconindia.com
Website : www.vectorbiotekindia.com | CIN No. : U33112GJ2002PTC040978


UA	4.40	3.70-5.10	4.72
ALB	4.25	3.50-5.00	3.7
UREA	32.20	24.00-40.40	31.13
TP	6.70	5.50-7.90	6.8
AMY	59.50	51.40-67.60	53.2
CA	9.70	7.90-11.50	9.4
CRE	1.85	1.30-2.40	1.47
BIL-T	1.20	0.90-1.50	1.04
TG	184.50	165-204	195.15

Note: Performance of the parameters are subject to traceability of Reagents, Manufacturer or Supplier.

Tested By:



Checked By:

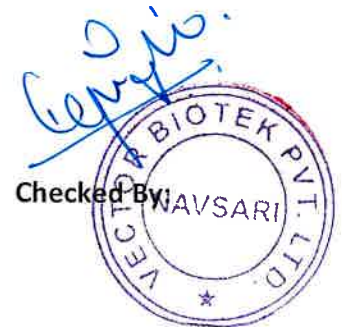



VECTOR BIOTEK PVT. LTD.

FACTORY & REGISTERED OFFICE :
425, NEW GIDC, KABILPORE, NAVSARI - 396 424 (GUJ) INDIA
Toll Free : 1800-270-1027 | E-mail : vector@beaconindia.com
Website : www.vectorbiotekindia.com | CIN No. : U33112GJ2002PTC040978

Operational Qualification

MODEL: B AUTO 200

Sr. No: AES2K0WH01023

Version: V4.8.3.0B3

Date: 08-06-2022

Operational Qualification Protocol run at Reliable Diagnostic centre, Chomu, Jaipur, Rajasthan.

#	Description	Status
1	Blank/Optics	OK
2	Assays(Parameter) Calibration	OK
3	Assay (Parameter) Control – BIORED	OK
4	Assays(Parameter) Samples(Patient)	OK
5	Programming and operational training completed	OK
6	Instrument operating training completed	OK

- Error msg. list in case of hardware issue: Refer Operational Manual page no. 75

I confirm that the above specified analyzer B AUTO 200 (Automatic Bio-chemistry Analyzer) is installed properly and its performance is within the specification.

Tested By:



Checked By:

