

Installation Qualification and Checklist

Mindray BS-430

Customer Name : Dr. Pankaj Diagnostics & Research Centre Pvt. Ltd.
Customer Address : B-27, Sachiwalay Colony, 200 m south of Kendriya Vidyalay Main Gate,
Kankarbagh, Patna, Bihar - 800020
Contact Person : Mr. Deepak Kumar
Instrument Model : BS-430 Chemistry
Analyzer Serial No. : YX-14005478

Site survey done and pre-installation checklist was made and found that the site is complying to all the Requirements The instrument packaging was checked and opened for installation.

Initial Inspection of the unit carried out and the details are as follows:

System Condition Report:

System delivered in satisfactory condition and no external physical damage observed on the same, Package was kept in good Condition as per the directional indicators like not tilt, indicating the system has not been subjected to mechanical shocks or stored in any manner, so as to cause any damage to the same.

Found all the required accessories are present.

- 1.
- a. Any Physical Damage to the shipment : Yes No
 - b. Accessories as per packing List : Yes No
2. Environmental Conditions.
- a. Air Conditioned Environment Available : Yes No
 - b. Room free of Rodents/ Insects/ Pests : Yes No
 - c. Humidity is less than 80 % : Yes No

3. General Cleanliness Around Site : Acceptable Not Acceptable

4. Power Requirements.

a. Uninterrupted Power Supply Available.

- UPS Rating -No star rating. 15KVA Online UPS
- Type of batteries - 12V-100AH x 30
- Back up Time - 2 hours
- Incoming AC Power: 229V
- Earth Voltage - 3V

5. Verified all Instruments Components and Accessories : Yes

6. Installed all Accessories as per the Instrument Set - Up Procedure Below : Yes

- a. Inspect all shipping cartons for possible damage. Check to make sure all kits necessary for the installation of the instrument are present.
- b. Assembled Sampler & Photometric Module, User PC, Keyboard.
- c. Installed the System Hardware, Computer & Printer.
- d. Checked the Electrical Power supply and Voltage.
- e. Load Supplies on the system (Water , Wash Solution, Cleaning Solution)
- f. Checked all cable connections & tubing's of Sample, Reagent Syringes.
- g. Checked all Electronic Boards are properly plugged in.
- h. Aligned the Wash Nozzle Dispense, Aspirate Probes, and Reagent, Sample Probe.
- i. Installed the current Software Version.

Installation Certificate

This is to certify that the BS-240 Chemistry Analyzer Instrument Serial number **YX-14005478** have been successfully Installed and commissioned at as per the Installation Procedure & Checklist.



PRIMAN INSTRUMENTS PVT. LTD.

M31112041N

Quality Product and Service for Medical Laboratory & Biotech Instruments, Consumable & Reagents "An ISO 9001:2008 Certified Company"

SAGAR GALAXY, IInd Floor, LSC, Mayur Vihar - II, New Delhi -110091
e-mail : priman@primanindia.com / email : admin@primanindia.com

Tel.: 22784431, 22784432,
Website : www.primanindia.com

Name: MD ALI MEHDI
Designation: Service Engineer
Date: 01rd-March-2022

Name:
Designation:
Date:

Thanking You
Yours Faithfully,
For Priman Instruments Pvt. Ltd.

Authorized Signatory