Roche Diagnostics India Pvt. Ltd. Field Service Report

Roche

Case number: CAS-0016843895

Order Number: WO-03362122 | Ser. Report Created Date: 26/03/2023

Instrument Details

Customer details

Internal Instrument Name:

Instrument/Module: COBAS INTEGRA 400 plus

Customer Number: 0052614393

Serial Number: 420471

Customer Name: M/s Pathkind Diagnostics Pvt Ltd

Street Address: Near HDFC Bank ,Shadatpura

Zip: 275101 - Mau

Additional Details

Contact Name: Rajesh

Contact Phone: +919547874563



Service Activity Code: PM Visit

Purpose of Visit

PM COBAS INTEGRA 400 plus Major

Performed Activities

Check and clean all PCB's, Clean ROTOR, Incubation area, washing stations, Clean probes and robotic transfer head and lubricated properly

Clean Analyzer, air filter and internal reservoir

Check and replace new KIT MAINTENANCE I400/400plus GMMI-05670713001, replace PM kit is provided by Roche under FOC

Restart system along with all required consumable

Perform all due maintenance and priming

Perform air/water calibration,data found ok

Perform all parameters calibration and QC

Results found ok

Per samples results found ok

M/C Status ok working satisfactory

PM Major done successfully



Time Report

Category	Start	End	Hours	Invoice Type
05913616001-Service labour time	26/03/2023 2:45 pm	26/03/2023 6:30 pm	3.75	Contract

Travel

Value	Hours	
Travel	3.9	filling the security during the film of the security of the se

Spare Parts

Roche Diagnostics India Pvt. Ltd. Field Service Report



Case number: CAS-0016843895

Order Number: WO-03362122 | Ser. Report Created Date: 26/03/2023

	Part Number	Description	Quantity	Invoice Type	
1 1 1 1 1 1 1 1 1 1 1 1 1 1 1 1 1 1 1	05670713001-KIT MAINTENANCE 1400/400PLUS KIT MAINTENANCE 1400/400PLUS		1	Contract	

Roche Diagnostics India Pvt. Ltd. Field Service Report



Case number: CAS-0016843895

Order Number: WO-03362122 | Ser. Report Created Date: 26/03/2023

Signature

Customer Signature Rajesh

Roche support Rajnish Dwivedi

Customer Signature

Roche Signature

The customer acknowledges the service intervention as performed in accordance with Roche recommendations:

- This Service report has been signed by the authorized representative of your organization.

- Discrepancies if any must be intimated within a period of seven (7) days of the receipt of the email, in the absence of which the report will be final and conclusive.

- This report shall be governed by the laws of India and the courts at Mumbal shall have the exclusive jurisdiction to deal with the disputes arising hereunder.

INTEGRA 400 plus

System Version:

3.6.2.1904

Cul Version:


3.6.2

Cul Build:

1904

Mar 26, 2023 / 16:33

System Serial Number: 420471

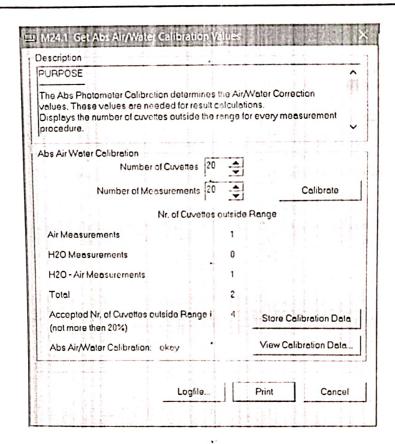




INTEGRA 400 plus

System Version: 3.6.2.1904 Access and Cul. Version and Cul. Version and Cul. Version and Cul. Build: 1904 Access and Cul. Version and Cul. Ver

System Serial Number: 420471 Mar 26, 2023 / 16:35

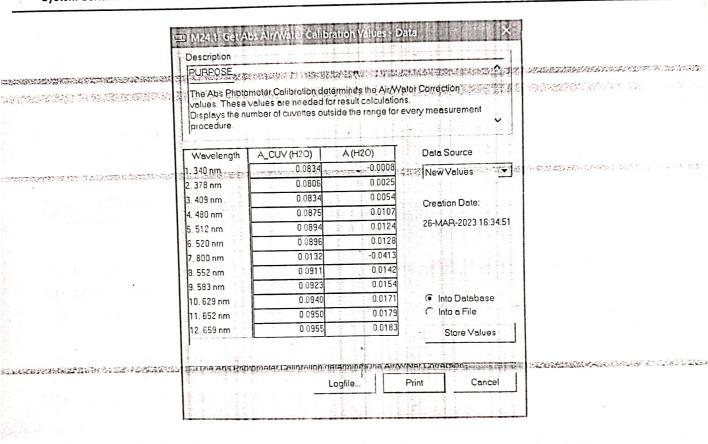



To Whom It May Concern

For ISO 15189:2012 and ISO 15189:2014 accredited Laboratories — requirements regarding "Calibration & Verification Procedures" [1]

All In vitro Diagnostics Products which are manufactured and distributed by Roche Diagnostics GmbH and for which a Free-Sales-Certificate is issued, are CE-marked.

The In-Vitro-Diagnostics Directive of the European Union [2A.] which is currently switching to IVD Regulation 2017/746/EU (final timeline: May 26, 2022) [2B.] requires for all CE marked products that the manufacturer assures compliance of the products with the requirements of the mentioned directive or regulation. This means that all processes in development and manufacturing of Roche Diagnostics GmbH products are guided by a Quality Management System. Our Quality Management System is in compliance with the requirements from ISO 13485:2016 [3] and 21 CFR Part 820 [4]. The mentioned regulations and standards require that the production systems and measuring devices used are qualified and the manufacturing and test procedures are validated. This status has to be assured by scheduled maintenance and by regular qualification resp. validation reviews and updates.

All physical quantities, calibrators and controls used in Roche Diagnostic systems are fully traceable to certified standards or reference materials. The performance of all In-vitro diagnostics systems of Roche Diagnostics GmbH at the customer site is assured if regular Quality Control measurements, cleaning and maintenance procedures as described in the instructions for use or service documentation are performed. By having controlled internal procedures and by running the tasks required in the respective user documentation, all In-vitro diagnostics systems of Roche Diagnostics GmbH will be performed as specified during their defined lifetime.

Additional calibration or verification procedures are NOT required by the user in order to assure the specified performance of every system of Roche Diagnostics GmbH. Only if a user deviates from these manufacturer's recommendations, the user have to establish site-specific calibration and verification procedures as part of his accreditation process.



- [1] ISO 15189:2012/ ISO 15189:2014 Medical laboratories Requirements for quality and competence
- [2] A. Directive 98/79/EC of the European Parliament and of the Council of the 27 October 1998 on vitro diagnostics medical devices;
 B. IVD Regulation 2017/746/EU of the European Parliament and of the Council of 5 April 2017 on in vitro diagnostic medical devices and repealing Directive 98/79/EC and Commission Decision 2010/227/EU
- [3] EN ISO 13485:2016 Medical devices Quality management systems-Requirements for regulatory purposes
- [4] CFR Part 820, Quality System regulations 21 Regulations on medical devices

Mannheim, 10. August 2021		
Sincerely,		
Roche Diagnostics GmbH		
i.V./on behalf of the company	ppa/on behalf of the company	
Andrea Weber	Ralf Zielenski	
Manager Global Regulatory Affairs	Head Q&R Compliance, PRRC RDG	

Centralised and Point of Care Solutions

Centralised and Point of Care Solutions