

**Installation Qualification, Performance,  
Qualification and Operational  
Qualification**

**YUMIZEN H550**

(Serial no: 207YAXH03883)

Fully Automated Hematology Analyzer

For

**LUPIN DIAGNOSTICS DL LAB BHOPAL**

**HORIBA**  
Medical

---

#246, Okhla Industrial Estate, Phase III, New Delhi 110020, India, Tel: 011 4646 5000.

Visit us: <http://www.horiba.com/in/>

---

*Nantel*

## **Table of Contents:**

### **Section 1 General Instructions**

### **Section 2 Installation Qualification**

- I. Installation Qualification.
- II. Installation Procedure.
- III. Installation Certificate.

### **Section 3 Operational Qualification**

- I. Operational Qualification.
- II. Operational Training Record.
- III. Operator Maintenance Protocol.
- IV. Operational Certificate.

### **Section 4 Performance Qualifications**

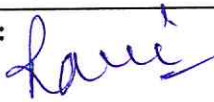
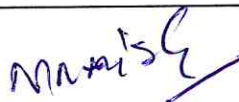
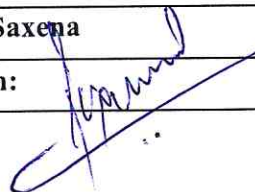
- I. Performance Qualification.
- II. Performance Certificate.

*Munish*

**General Instructions:**

- HORIBA India Pvt. Ltd. is responsible for installation of YUMIZEN H550, Fully 6 - Part Automated Hematology Analyzer, at Lupin Diagnostics lab Bhopal.
- An authorized HORIBA India Pvt. Ltd., representative will physically check the system and proceed for the installation.
- This installation protocol will be followed as specified by the manufacturer.
- Installation checks will also be performed to verify that the instrument has been installed with proper connections and utilities.
- After the installation of the system, Instrument calibration & QC will be performed by the authorized HORIBA India Pvt. Ltd. representative.
- An authorized HORIBA India Pvt. Ltd. representative will also perform the precision check on the system to check if it is as per the claim of the manufacturer.
- The results obtained for Calibration, QC & Precision checks will be verified by the qualified trained employee Lupin Diagnostics lab Bhopal along with an authorized HORIBA India Pvt. Ltd.,
- On completion of the Installation all the necessary documents of the System checks will be used to evaluate the instrument installation in accordance with the manufacturer's protocol and intended use.
- An authorized HORIBA India Pvt. Ltd., representative will verify the documents of the system checks and approve the same.
- Successful completion of this protocol will verify that this instrument has been installed in accordance with the intended usage.

**Report Sign Off**

Prepared by:	HORIBA Medical - HORIBA India Pvt. Ltd.		
Name:	Mr. Ravindra Singh Chouhan		
Title: Enginner Support.	Customer	Sign: 	Date: 24.11.2022
Reviewed by:	LUPIN Diagnostics Lab Bhopal		
Name:	Mr. Manish Kumar Patel		
Title: Lab Manager		Sign: 	Date: 24.11.2022
Approved by:	LUPIN Diagnostics Lab Bhopal		
Name:	Dr. Priyanka Saxena		
Title: Pathologist		Sign: 	Date: 24.11.2022

